

# Suction Unit **SG-10**

Single Workbench Suction Unit









#### Dear customer,

Thank you for choosing a product from the Wassermann range. Wassermann Dental-Maschinen incorporates the highest standards of quality and the latest technology. In order to enjoy maximum performance and years of trouble-free operation, please read this user manual carefully before you connect this device and start work, and operate the device according to the recommended guidelines. The operation safety and the functionality of this device can only be guaranteed if you follow both the general safety guidelines and the applying laws to prevent accidents as well as the precautions given in this user manual. We are not liable for any damages which occur due to inappropriate usage or faulty operation of this device.



Make sure that anyone using this device has read and understood this user manual.

Keep this user manual in a safe place where it can be referred to as required at any time.

 $\epsilon$ 

The unit complies with the relevant EU guidelines.



The unit is subject to the EU guidelines 2012/19/EU (WEEE Directive).

#### Company address:

Wassermann Dental-Maschinen GmbH

Rudorffweg 15-17

21031 Hamburg, Germany

Phone: +49 (0)40 730 926 -20/ -24 Fax.: +49 (0)40 730 37 24

export@wassermann-dental.com

www.wassermann.hamburg

www.facebook.com/WassermannDentalmaschinen



# **Contents**

1	Features	4
2	Safety Symbols used in this Manual	4
3	Safety Guidelines	5
4	Responsibility for Operation or Damage	5
5	Application	6
6	Before Starting	6
	6.1 Transport	6
	6.2 Installation	7
	6.3 Storage	7
7	Installation/ Start-up	7
8	Operation	8
	8.1 General Operating Instructions	8
9	Troubleshooting	9
10	Care and Maintenance	10
	10.1 Cleaning	10
	10.2 Maintenance	10
	10.2.1 Replacing the filter	10
	10.3 Repairs	11
	10.4 Spare Parts	11
	10.5 Service Hotline 0049 (0)40 730 926 -20/ -24	11
	10.6 Scope of Delivery/ Accessories	12
	10.6.1 Use of the Filter Extension Module	13
	10.6.2 Operating with Activated Carbon Filter Slot	14
	10.7 Warranty	15
11	Technical Data	15
12	? Technical Drawings	16
	12.1 Construction SG-10 (incl. Activated Carbon Filter)	16
	12.2 Construction Module Activated Carbon Filter (REF: 140424)	17
13	B Disposing of the Unit	18
	13.1 Information on Disposal for Countries within the EU	18
14	EU Declaration of Conformity	19



#### 1 Features

The suction unit **SG-10** is used to extract dental dust at the workplace and to extract polishing dust and fine blasting units.

- Universal suction unit works with the low pressure principle
- Single workbench suction unit
- Power control is infinitely variable using the control element on the housing (an external control element can be retrofitted, accessory item no.140408)
- Modular filter system for different uses
- Innovative filter concept, easy-to-replace filter system
- Filter saturation display
- Other filters available: from the activated carbon filter to the high-volume plaster pre-filter
- Suitable for continuous operation
- High-quality finish
- Free from wear and maintenance-free
- Recommended version for our fine blasting units

# 2 Safety Symbols used in this Manual



# Warning!

This is a warning of risk situations and dangers.

Failure to observe this warning could be life-threatening. These warnings has to be observed.



#### Information!

This symbol draws your attention to specific features that has to be observed.



# 3 Safety Guidelines

Configuring and operating this equipment requires *precise knowledge* and *observance* of the instructions in this *user manual*. The equipment is designed only for its intended application.



#### **WARNING:**

Servicing and repairs should be carried out only by authorised specialists. Disconnect the power plug before starting any maintenance work.



Make sure that the equipment is connected to the correct power source.



Do not operate the unit without a fan guard or a filterbag! Do not operate the **SG-10** in an explosive atmosphere! Do not apply suction to any liquids.

# 4 Responsibility for Operation or Damage

The responsibility for operating the device lies exclusively with the owner or user if said device is incorrectly serviced, maintained or altered by persons not employed by an authorised dealer or if the device is used in a manner contrary to its specified purpose. The unit has to be maintained and operated in accordance with this user manual. Wassermann Dental-Maschinen GmbH is not responsible for damage arising from the **nonobservance** of these instructions.

Warranty and responsibility provisions contained in the sales and supply conditions of Wassermann Dental-Maschinen GmbH are not extended by these instructions.



# 5 Application

The suction unit **SG-10** is versatile in use, e.g. at dental technician workplaces, with blasting units (e.g. Cemat NT, Puna) or with a polishing splash guard.

Maximum air extraction at low air speed is guaranteed. Only respirable suspended solids are extracted.

The benefits this provides are practically silent operation of the fan motor and quiet running due to the low flow speed of the extracted air. The slow air flow means that even during work over longer periods, the hands do not chill.

The **SG-10** has an rotary potentiometer for fan control and a pilot lamp to indicate filter saturation on the housing.

Additionally, the unit's modular structure permits operation with activated carbon and/or a pre-filter setting, e.g. for larger quantities of plaster. Furthermore, a specially developed suction hood is available for the dental technician workplace, hence optimising the suction performance.

Only use the device for this type of application.

# 6 Before Starting

# 6.1 Transport

Before transporting the unit, ensure that it has been unplugged from the power socket. Make sure that it is packed correctly in order to avoid accidental damage.



Be sure to check for any transport damage when unpacking the goods. Note down any damage if found.



#### 6.2 Installation

Open the box, remove the packing materials, and carefully lift out the device and accessories. Check the included accessories.

The device has to stand horizontally on a steady and even surface.



Install the device in a place where it will not block the working area and the functionality (take the dimensions into account). Make sure that there are about 10 cm of free space on rear side to guarantee air circulation (heat dissipation).



Do not install the unit outdoors or in places without proper ventilation.

Before start-up, be sure the device reaches room temperature.

# 6.3 Storage

The unit location is very important when it comes to workplace safety, even if it is only to be set up there for a short period. The room should be dry, well ventilated and vibration-free. An even temperature and wooden supports also help. The unit should not be stored or set up outdoors. If the unit is to be stored for an extended period, protect it from moisture and dust.

# 7 Installation/ Start-up

- 1. Remove the supply line from the filter below the lid.
- 2. Make an electric connection.
- 3. Insert the power plug into the socket, making sure that the mains and the unit operate on the same voltage.



# 8 Operation

# 8.1 General Operating Instructions

All instructions for using the unit, whether in verbal or written form, are based on our own experience and experimentation and can only be regarded as guidelines.

- 1. The unit can be switched on and off by pressing the rocker switch on the supply line.
- 2. A rotary potentiometer for fan control is located on the left side of the housing. Start the fan and set the desired level by turning the control element. It can also be used to <u>briefly</u> stop the fan.
- 3. Use the rocker switch to deactivate the **SG-10** during longer interruptions in work.



You <u>have to</u> press the rocker switch on the supply line in order to deactivate the unit!

The control element can be used to stop the fan, but the unit itself is not switched off!

1

**Delayed fan start-up is a feature of this series** and permits maintenance-free operation and a longer service life.



# 9 Troubleshooting

Fault	Cause	Solution
Filter saturation  – red lamp glows	Full filter	Replacing the filter (see 8.2.1)
Motor control does not work	Damaged potentiometer (speed control)	Please contact our service department
Full filter – but red lamp does not glow	Damaged lamp	Please contact our service department
	Damaged differential pressure switch	
Empty filter – but red lamp already glows	Damaged differential pressure switch	Please contact our service department



If the above recommendations do not solve the problem, contact your dental depot or our service department.



#### 10 Care and Maintenance

# 10.1 Cleaning



Disconnect the power plug before starting any maintenance work.

The identification plate has always to be kept in easily legible condition and has not to be removed.



Remove external dirt from time to time with some form of cold cleaner.

Use only cold cleaners to avoid damaging the paintwork or removing the lettering.

The equipment should be cleaned at regular intervals to ensure trouble-free operation.

It requires only normal cleaning (sponge, damp cloth, mild detergent) and no further chemical additives.

#### 10.2 Maintenance

The unit does not require any servicing. Just make sure that the device is kept clean.

# 10.2.1 Replacing the filter

Replacing the filter is part of the maintenance work.

Please replace the filter in the event that the suction performance falls noticeably, and certainly when the pilot lamp lights up.



Regular filter replacement is necessary to ensure maximum dust extraction. **The red pilot lamp** for filter saturation was set to maximum suction performance. It simply **indicates that the filter should be replaced**; it does not deactivate the fan.



A <u>disposable micro fine dust filterbag</u> is fitted; it must be replaced when dust saturation (**red pilot lamp**) is reached.

To do this, open the fasteners on the right and the left, remove the lid, take out the filterbag and wrap in the adhesive foil enclosed with the spare filter. Insert the new filter, close the lid.



Depending on their use, activated carbon filters should be replaced when necessary, but after no later than 4 to 6 months. The filter cassette dust class M (secondary filter) should be replaced depending on its use (recommendation: use for no more than 12 months).

# 10.3 Repairs



Servicing or repairs to the unit has only to be carried out by qualified technicians. Only original spare parts are to be used. Responsibility for the product is voided if unauthorised persons alter it or if inappropriate components are installed.

# 10.4 Spare Parts

If necessary please contact our service hotline phone.

# 10.5 Service Hotline 0049 (0)40 730 926 -20/ -24



# 10.6 Scope of Delivery/ Accessories

Included parts	Item no.
SG-10 basic version	140988
Suction unit SG-10 base	140980
Supply line with switchable shock proof plug	140022
Micro fine dust filterbag (yellow) small, dust class M	

Included parts	Item no.
SG-10 activated carbon version	140987
Suction unit SG-10 base	140980
Supply line with switchable shock proof plug	140022
Micro fine dust filterbag (yellow) large, dust class M	610074
Filter extension module (without filter)	140410
Module activated carbon filter (with filter)	140424

Accessories:	Item no.
Spare micro fine dust filterbag (yellow) small, dust class M, 1 set = 4 pieces	611010
Spare filterbag for plaster (white) small, class 1 / class 2, 1 set = 4 pieces	611018
Spare micro fine dust filterbag (yellow) large, dust class M *	610074
Spare filterbag for plaster (white) large, class 1 / class 2 *	610073
Spare filter cassette, dust class F, cpl.*	140435
Spare activated carbon filter, cpl.*	140436
Module filter add-on (without filter)	140410
Module activated carbon filter (with filter)	140424
External speed regulator for fan control, can be retrofitted	140408
(accessory SG-10 #00168724 ff.)	

<sup>\* (</sup>only SG-10 activated carbon version)



Item numbers for setup change SG-10 basic into activated carbon version: 140410/ 610074/ 140424



#### 10.6.1 Use of the Filter Extension Module

In order to fit a <u>larger filterbag or the activated carbon filter</u>, please use the filter extension. To do this, open the fasteners on the right and the left, take off the lid, insert the filter extension and close the fasteners.

Insert the new filter (e.g. plaster filterbag, long) into the filter extension, replace the lid and close using the fasteners on the right and the left.





# 10.6.2 Operating with Activated Carbon Filter Slot

In order to insert the activated carbon filter unit into the device, please first remove the old filterbag (as described under item 8.2.1).

After this, insert the <u>slot already fitted with filters</u> into the unit and fit the intermediate frame.

Then fit the filter extension to the unit (see item 8.5.1), insert the pre-filter (filterbag dust class M, long 340 mm) and close the lid.

In order to replace the activated carbon filter while using the **SG-10**, please remove the activated carbon filter slot from the unit and replace the used filters.



When fitting, resp. replacing the activated carbon filter, please initially press the 'filter cassette dust class M' (white folded filter) into the slot for activated carbon filter operation in the direction indicated by the arrow (displayed on the filter cassette). The activated carbon module cassette (black cassette) is then fitted.







# 10.7 Warranty



The warranty period for our equipment is 12 months. If faults occur within the warranty period, contact your dental depot or get in touch directly with our service department.

Your equipment should only be operated in perfect condition. If faults occur which could harm operators or third parties, the unit should not be used until it has been fixed.

This warranty does not cover damage caused by improper use, external mechanical causes, transport damage or interference with the unit by unauthorized persons.

#### 11 Technical Data

SG-10 basic version Item no.: 140988

SG-10 activated carbon version <u>Item no.: 140987\*</u>

Voltage<sup>(\*)</sup> 220–240 V / 50/60 Hz

Power consumption 0.75 A
Output 110 W

W x H x D 348 (without plug) x 423 x 270 mm

348 (without plug) x 672 x 270 mm\*

Weight 8.2 kg depends on version

Sound level < 70 dB (A)

Suction capacity 600 m<sup>3</sup>/h (free-blowing)

270 m<sup>3</sup>/h (operating state)

Extractor nozzle Ø 75 mm exterior

<sup>(\*)</sup>Other voltages on request



The noise level of the unit amounts to  $\leq$  70 dB (A).

The emission value may be higher, depending on the number and the operating conditions. Please take suitable protective measures (e.g. ear protectors).




Technical changes reserved.



# 12 Technical Drawings

# 12.1 Construction SG-10 (incl. Activated Carbon Filter)


1	REF: 140410	Filter extension module (without filter)
2	REF: 140424	Module activated carbon filter (with filter)
3	REF: 140980	Suction unit SG-10 base
4	REF: 310035	Identification plate (serial number etc.)
5	REF: 610074	Spare micro fine dust filterbag (yellow) large, dust class M

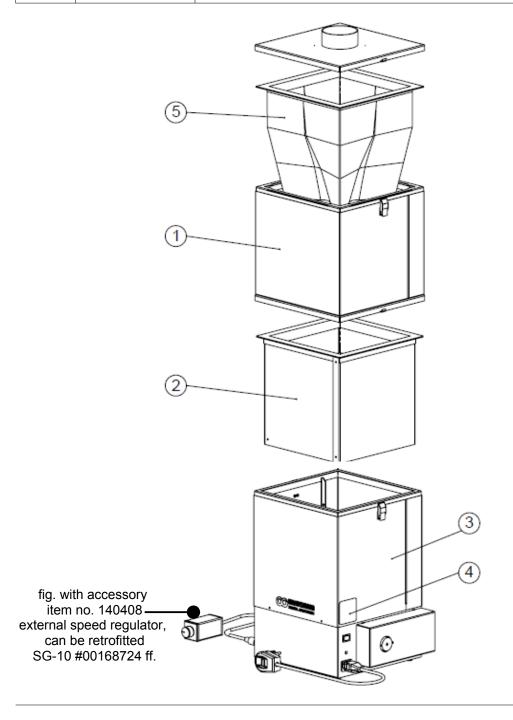



# 12.2 Construction Module Activated Carbon Filter (REF: 140424)

1	REF: 140426	Filter casing, cpl. SG-10
2	REF: 140427	Filter frame, cpl. SG-10
3	REF: 140435	Spare filter cassette, dust class F, cpl.
4	REF: 140436	Spare activated carbon filter, cpl.

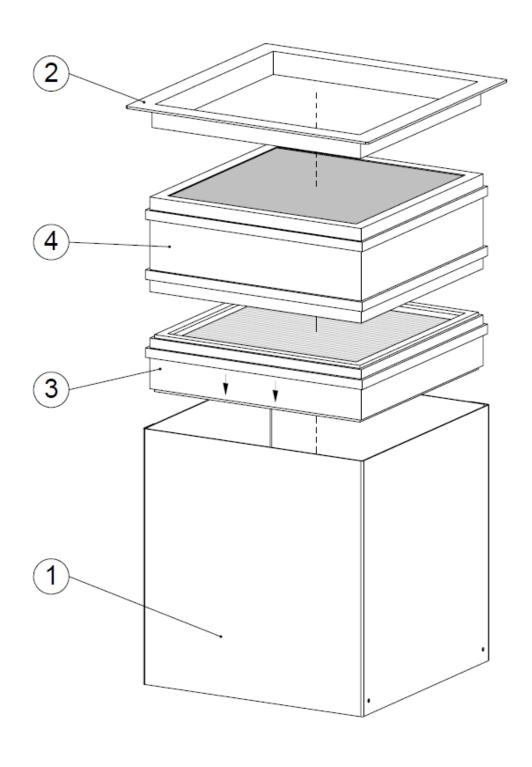



# 13 Disposing of the Unit

The unit has to be disposed by an authorized recycling operation. The selected company has to be informed of all possibly health-hazardous residues in the unit.

# 13.1 Information on Disposal for Countries within the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste.

For more information regarding proper disposal please apply at your local authoritie.



# 14 EU Declaration of Conformity

in accordance with 2014/35/EU (Low Voltage Directive) and 2014/30/EU (Electromagnetic Compatibility Directive) and 2006/42/EC (Machinery Directive) and 2011/65/EU (RoHS Directive)

Manufacturer: WASSERMANN Product description: Suction Unit for

Dental-Maschinen GmbH dental

Rudorffweg 15-17 applications

21031 Hamburg

Germany

Model: SG-10 basic version Item no. 140988

SG-10 activated carbon version Item no. 140987

Applicable DIN EN 61010-1 standards: DIN EN 61000-6-3

DIN EN 61000-6-1 DIN 45635-1

DIN EN 60335-1

Hiermit wird bestätigt, dass die oben bezeichnete Maschine den genannten EU-Richtlinien entspricht. Diese Erklärung wird ungültig, falls die Maschine ohne unsere Zustimmung verändert wird.

This is to confirm that the above mentioned machine complies with the described EU guidelines. This declaration becomes invalid if the machine is modified without our approval.

Cette machine est conforme aux normes en vigueur d'Union européenne. Cet avis est nul et non avenant si cette machine est modifiée sans notre accord.

Esta máquina, anteriormente mencionada, cumple con los limites requeridos por el reglamento UE. Ahora bien, esta declaración quedará invalidada en caso de realizar modificaciones al aparato sin nuestra aprobación.

Hiermee wordt bevestigd dat bovengenoemde machine voldoet aan de voorgeschreven EU normen. Deze verklaring verliest geldigheid als er zonder onze uitdrukkelijke toestemming wijzigen aan de machine worden aangebracht.

Place, date: Hamburg, 2020-09-03 Company stamp:

Jue Ware

Signature: \_\_\_\_\_\_Sven Wassermann (Managing Director)



Wassermann Dental-Maschinen GmbH Rudorffweg 15–17 · 21031 Hamburg · Germany Phone: +49 (0)40 730 926-20/ -24 · Fax: +49 (0)40 730 37 24 export@wassermann-dental.com · www.wassermann.hamburg

