HowTo - Look here

Schleifscheibenwechsel

HSS-88

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GB **Included in delivery/ Accessories**

Included in delivery	Item no.
Spare diamond grinding disc, incl. puller and grease	168063
Spare corundum grinding disc, incl. puller and grease	168024

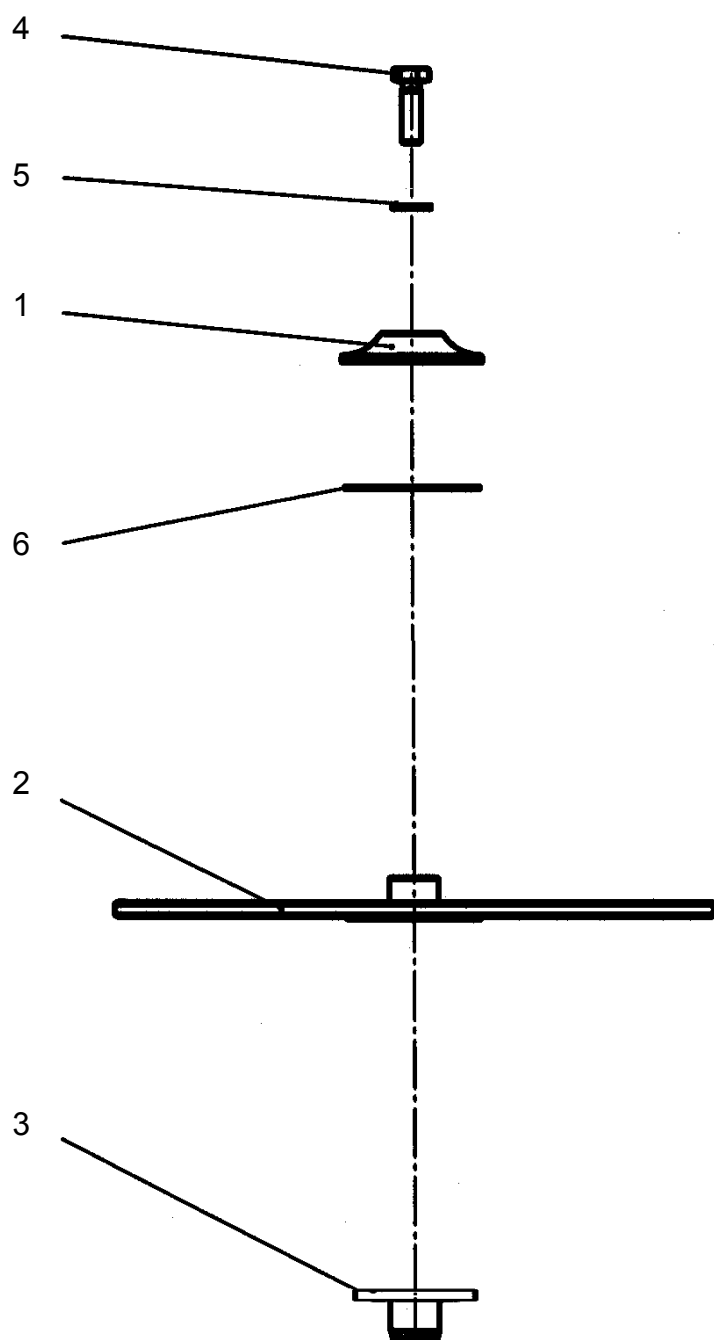
Accessories	Item no.
Sealing grease for trimmer, 25 ml	629002
Puller for grinding disc change	164026

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GB HSS-88: Replacement of the Diamond Grinding Disc and Corundum Grinding Disc

WARNING:

Disconnect the power plug before starting any maintenance work.

1. Loosen the star handles to take off the front cover.
2. Loosen the central screw on the axle counter-clockwise and hold the grinding disc.
3. If necessary loosen the thrust piece by sticking a screwdriver between the thrust piece and the grinding disc and take it off together with the PVC washer.
4. Screw on the puller which is supplied with the replacement grinding disc onto the now visible thread. Then you pull off the grinding disc down from the axle by screwing in the central screw.
5. Clean the shaft stub and brass bushing thoroughly and grease them slightly with the lubricant supplied with the replacement grinding disc. Put on the grinding disc by slightly turning. Put on the thrust piece and the PVC washer and screw on the central screw.
6. Check if the disc runs smooth by pushing it with the hand.
Advice: Use a piece of chalk and hold it steady to the disc while you twist it with the hand. Through this you could notice a possible hit which you could fix by pressing with the heel of hand.
Attention!: Please do not use any tool for this! In case you cannot correct the hit please contact our service team.
7. Hold the grinding disc firm and screwing on the fixing screw tight. Check again for any hits. If you notice any hits please follow step 5.
8. Clean the contact surfaces of the housing parts and seal them with the grease supplied by applying a line of grease on one part of the housing. Do not spread it.
9. Put back the front plate and fasten the star handles tightly. First the bottom star handles, then the top star handles. Now spread the grease and remove any grease.

Please also use the delivered grease for greasing the windings!

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Wassermann Dental-Maschinen GmbH